

Hydra Research


Assembly: Face Shield - M

How to assemble the Hydra Research Face Shield - M.

Written By: John Kray



Step 1 — Before You Begin

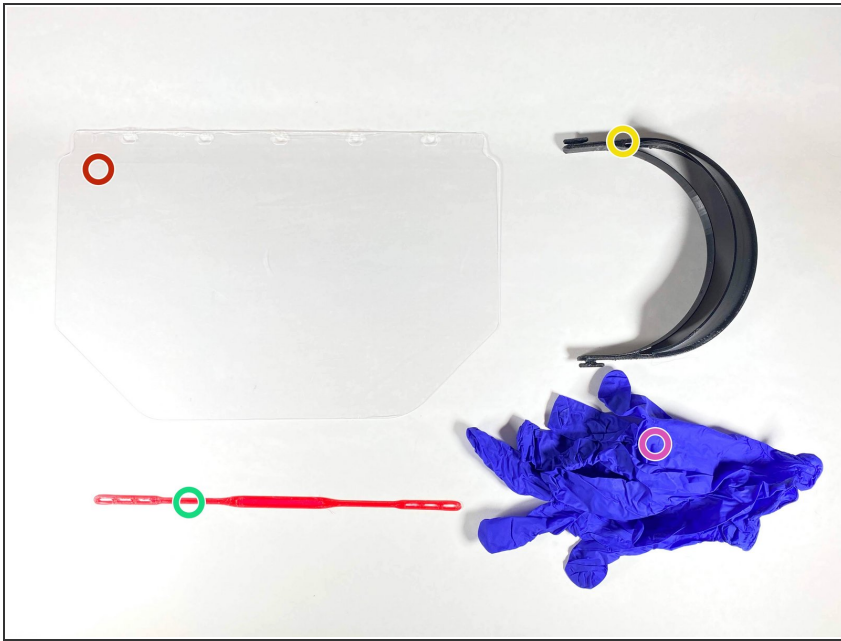


⚠ Use at Own Risk – Not Intended For Any Use Requiring Regulatory Approval.

i If possible wait 72 hours after receiving your face shield to open and assemble it. **WASH HANDS BEFORE STARTING!**




- Make sure you have the following before you begin.
- Clear Foil
- Face Shield Head Piece. We are constantly improving the design, the look may vary slightly from the pictures, but functionality will remain essentially the same.
- Headband. May be be flexible rubber or elastic.
- Gloves (if possible)

Step 2 — Prepare Foil

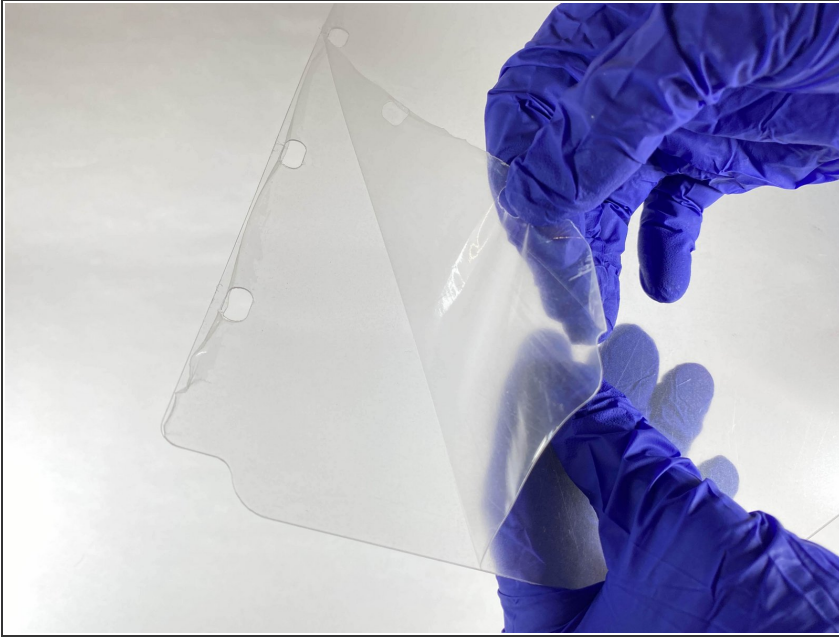


- Remove the protective film from BOTH sides of the clear foil.

Step 3 — Attaching the Foil

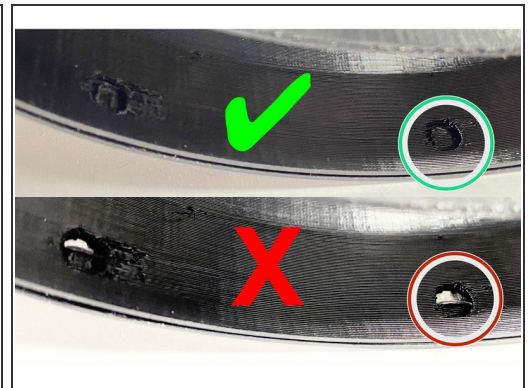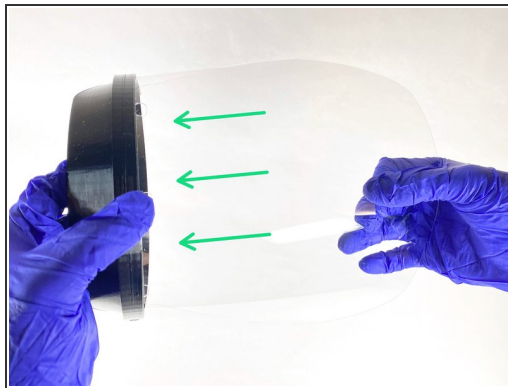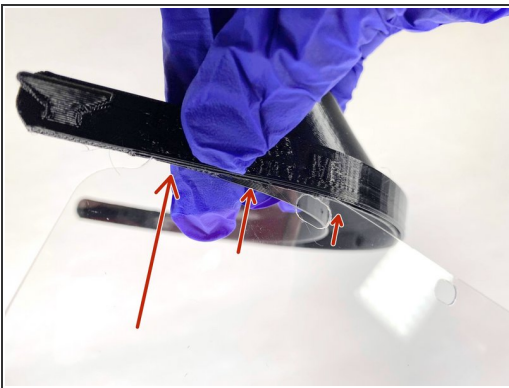


- Insert the foil into the slot that goes around the front band of the head piece.
- Start at one side and work your way around, seating the foil.
- Finally push the foil up into the head piece until all the clips engage.

 Please see that last image to check to make sure the foil is properly in place.

 Be careful when inserting the foil so that you don't break the headpiece. When clipping the foil into place, push on the headband, not the forehead cover part of the head piece.

Step 4 — Attaching the Headband

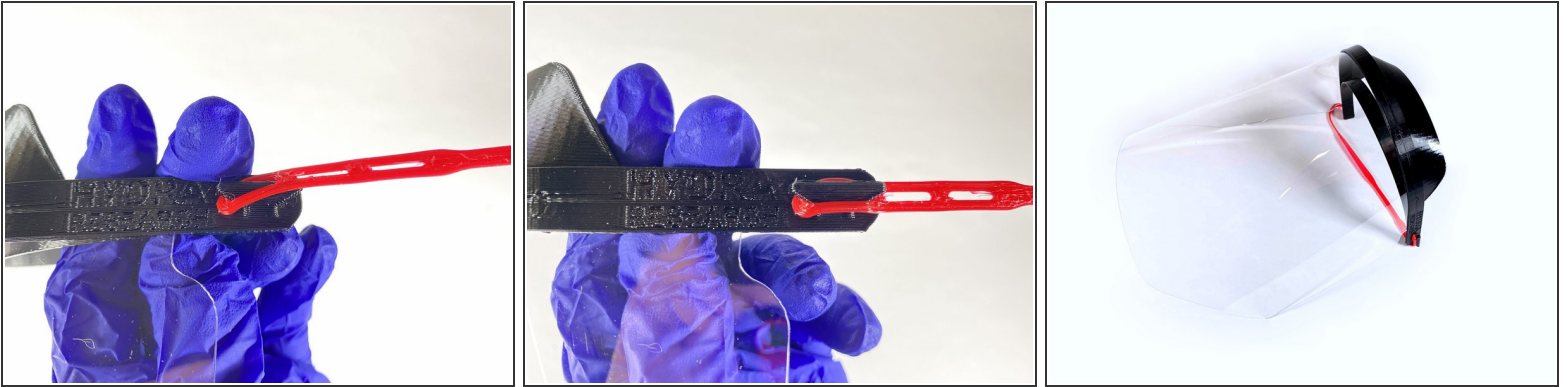

- Depending on our supply chain, you may receive a flexible rubber, or elastic headband.
- Hook the hole in one side of the headband over the hook on the head piece.
- Stretch the head band over the other prong of the hook
- Repeat on the other side.

⚠ Sanitize your face shield before use and wash hands again!

- You are all set, put your face mask on and stay safe!
- [Sterilization Guide](#) — Information on the sterilization of 3D printed face shields provided by Prusa Research. URL: tinyurl.com/twlpaub